Cardiac MRI Toolkit LA Segmentation Workflow

Salma Bengali, Alan Morris, Josh Cates, Rob MacLeod

CARMA Center

Email: salma.bengali@carma.utah.edu

NA-MIC Tutorial Contest: Summer 2014



This tutorial demonstrates how to use the LA Segmentation Workflow in the Cardiac MRI Toolkit Slicer extension.



To use this tutorial, you would need to have completed:

Data Loading and Visualization by Sonia Pujol, Ph.D., Harvard Medical School

http://www.slicer.org/slicerWiki/index.php/Documentation/4.0/Training



This tutorial requires the installation of the Slicer nightly binary and the tutorial dataset. These are available at the following locations-

Slicer download page: http://download.slicer.org/

Tutorial dataset: Cardiac MRI Toolkit Tutorial Data

http://wiki.na-mic.org/Wiki/index.php/ File:CMRToolkitLAWorkflowData TutorialContestSummer20 14.zip



This tutorial was designed for-





10.8.5





- This tutorial explains the usage of the LA Segmentation Workflow module which is part of the Cardiac MRI Toolkit Slicer extension
- The tutorial contains screen shots which will explain how to run each step







The extension can be downloaded using the Extension Manager, after installing the latest nightly binary of Slicer:

http://download.slicer.org/

The extension can be built manually after downloading our source code from the CARMA Center GitHub repo:

https://github.com/carma-center/carma_slicer_extension



Modules:	Regmentation Workflow 主 🚍 🧿		🔲 t - 🐻 🚲 🛼 + - 😫 👶	-s © :).000mm
Click the 'Data' button	yzed. Use the test data or your own images.				
OR, download two sample images	Download test MRA and MRI data				
Back	Next	0			
▼ Data Probe					







🔝 🚵 Modules: 🔍 🛃 LA Segmentation Workflow 💠 💻 🤤 🕗 🎕 🜌 🖉 📗 🛄 🕴 🔹 🖓 🥵 🚳 🤹
© K 👫 🙀
3DSIIcer ≫ S ↔ Axial ♥ Registered MRA ♥
Help & Acknowledgement
c. image neglisitation Positive the MAL image to the LGE-MPI image
Independition with Antilage to the Cole-Winh integer
Registered MRA: Registered MRA
Register MRA to LGE-MRI
▼ Data Probe
Registered MRA image is generated.



📩 🦗 🐜 Modules: 🔍 🏤 LA Segmentation Workflow 💠	=, Q, Q, @ @ @ @ Z <
3DSlicer	
Help & Acknowledgement	
3. Segmentation of endocardium	
Segment the endocardium using the Editor tools.	
 Create and Select Label Maps 	
Master Volume: Registered MRA	
Marce Volume: Registered MRA-Jabel	Set Set
Per-Structure Volumes	Select existing label map volume to edit.
💌 Edit Coloriod Labol Man	
	Registered MRA-label
	Apply Cancel Create New
"C "G	
Undo/Redo:	
Anitus Teal	Create a new label image
Label: tissue 1	tor the segmentation
	the state of the Press
Back N	ext O
▼ Data Probe	
1	
L	
B	



	🛍 🛗 Modules: 🔍 🛃 LA Segmentation Workflow 💠 💻 🧿 🕥 🖤 🚳 🎕 🖾 ⋌ 🛛 🥅 😫 🖛 🗛 🗛 🔶 🛃 🥐
	Image: State of the state
	Help & Acknowledgement
	▼ 3. Segmentation of endocardium
	Segment the endocardium using the Editor tools.
	Create and Select Label Maps
	Master Volume: Registered MRA
	Merge Volume: Registered MRA-label Set
	Per-Structure Volumes
	▼ Edit Selected Label Map
	Image: Constraint of the second s
Sele segr	ect the 'CarreraSlice' nentation tool
	C Back Next C
	L F B



🎰 📸 Modules: 🔍 🛃 LA Segmentation Workflow 💠 💻 🧿 🕥 🖤 🚳 🍓 🜌	K 🛄 🕯 - 🐻 趣 🜉 🔶 - 😫 🥐
0 🛛 🐂 R 🛊 🚃	\$: 43.061mm
3DSlicer	
Help & Acknowledgement	
▼ 3. Segmentation of endocardium	
Segment the endocardium using the Editor tools.	
▼ Create and Select Label Maps	
Meeter Volume: Registered MRA	
Merge Volume: Registered MRA-label Set	
	A CONTRACTOR OF
Edit Selected Label Map	
Undo/Redo:	
Active Tool: DefaultTool	
Label: tissue 1	
	A - The A
Back Next	
▼ Data Probe	
	One sta final LA segure substitue
	Create final LA segmentation
F	
в	



Modules: 🔍 🛃 LA Segmentation Workflow 💠 💻 🧿 🕥	I 🖤 🚳 🛳 🗶 🧮 🕇 🖣 🐻 👞 👞 🔶 🖷 🗳	
	© ⊠ H R Internet Street Stree	S: 45.561mm
3DSlicer		
Help & Acknowledgement		
3. Segmentation of endocardium Segment the endocardium using the Editor tools.	Select the LGE from	
▼ Create and Select Label Maps	the drop-down list	
Master Volume: Registered MRA		and the second
Merge volume: Hegistered MHA-label Set		
▼ Edit Selected Label Map		
Undo/Redo: 🕡 🐠	1. The states	
Label: tissue 1		
	Alt the second second	
Back Next		1 - Parameter 2
▼ Data Probe		
Red RAS: (169.0, 54.2, 45.6) Axial Sp: 2.5		
L P2_endo (50, 233, 25) 0 (0) F None () B L GE-MBI (50, 233, 25) 139		
D EGE-III (00, 200, 20) 100		



Modules: 🔍 🛃 LA Segmentation V	Vorkflow 🗘 💻 🧿 🕥	. 🖉 🎱 🎕 🗹 🎸	🛄 🕇 🗸 👦 🐜 🔶 🛛 🔯 🥐
/3DSlicer		⊘ ⊠ ≫ ∮ ີ ▲ Axial	S 45.561mm
Help & Acknowledgement			
 3. Segmentation of endocardium 			
Segment the endocardium using the Editor tools.			
 Create and Select Label Maps 			
Master Volume: Registered MRA		\	
Merge Volume: Registered MRA-label	Set		
Per-Structure Volumes		1	
▼ Edit Selected Label Map			
R 5 2 2 3 1 8 8			
		1050 80	
Undo/Redo:			
Active Tool: Default	Tool		
		84V 83	
Label: tissue 1	.	- 17 ///	
		2 45	
Back	Next	0	Clean up the accompatizion as
 Data Probe 			Clean up the segmentation as
Red RAS: (169.0, 54.2, 45.6) Axial Sp: 2.5			required
L P2_endo (50, 233, 25) 0 (0) F None ()			



We have demonstrated the use of our LA Segmentation Workflow module for the purpose of segmentation of the left atrium using gated MRAs and LGE-MRIs.





National Alliance for Medical Image Computing NIH U54EB005149



Thank You!